

# Niemann-Pick Disease Type A

Niemann-Pick disease type A is a rare genetic disorder. It is characterized by an enlarged liver and spleen, developmental disability, recurring lung infections, and early death. A person must have two variants in the SMPD1 gene in order to have this condition.

Overview Scientific Details

Jamie, you **do not have the variants** we tested.

You could still have a variant not covered by this test.



## How To Use This Test

**This test does not diagnose any health conditions.**

Please talk to a healthcare professional if this condition runs in your family, you think you might have this condition, or you have any concerns about your results.

[Review the Carrier Status tutorial](#)

[See Scientific Details](#)

## + Intended Uses

- Tests for **multiple variants** in the SMPD1 gene.
- To identify carrier status for Niemann-Pick disease type A.

## - Limitations

- Does **not test** for all possible variants for the condition.
- Does **not report** if someone has two copies of a tested variant.
- Does **not cover** other types of Niemann-Pick disease.

## 🌐 Important Ethnicities

- This test is most relevant for people of Ashkenazi Jewish descent.

## You are likely not a carrier.

This result may be less relevant for you because the variants that cause Niemann-Pick disease type A are rarely found in people of your ethnicity.




We ruled out the tested variants for Niemann-Pick disease type A.

These variants are most common in people of **Ashkenazi Jewish** descent.

You still have a chance of being a carrier for Niemann-Pick disease type A.

We cannot estimate your chances because this condition is rare and not well studied in your ethnicity.

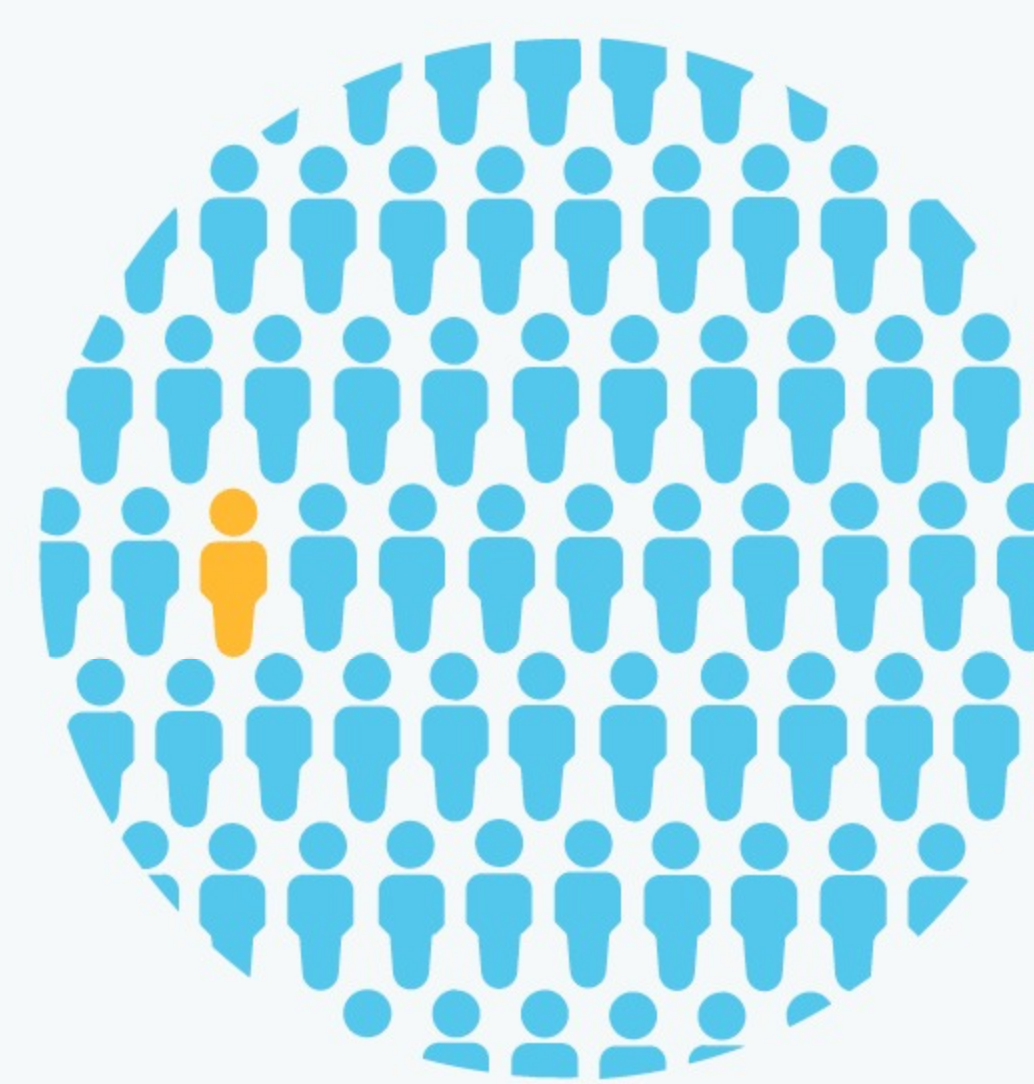

## About Niemann-Pick Disease Type A

**Also known as:** Acid Sphingomyelinase Deficiency

### 📅 When symptoms develop

Symptoms typically develop during infancy.

### 🧪 Typical signs and symptoms

- Enlarged liver and spleen
- Severe developmental disability
- Recurring lung infections
- Poor weight gain
- Death in early childhood

### 👥 Ethnicities most affected


This disease is most common in people of Ashkenazi Jewish descent.

### 🩺 How it's treated

There is currently no known cure. Treatment focuses on managing symptoms and preventing complications through physical and occupational therapy.

Read more at: [Genetics Home Reference](#) [GeneReviews](#) [National Institute of Neurological Disorders and Stroke](#)

## Consider talking to a healthcare professional if you are concerned about your results.

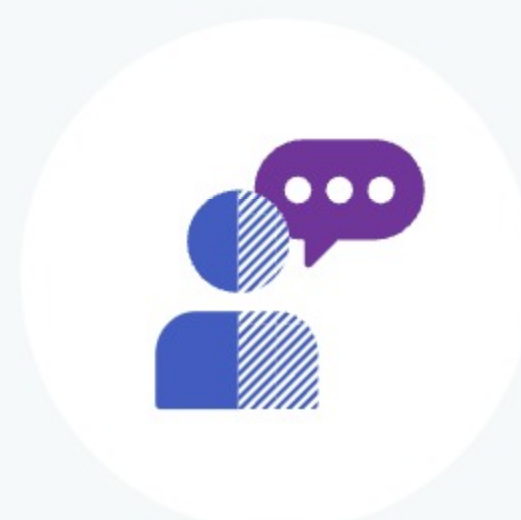

If you're starting a family, a genetic counselor can help you and your partner understand if additional testing might be appropriate.

Connect with a GC



Share your results with a healthcare professional.

Print report



Learn more about this condition and connect with support groups.

Learn more



Give the gift of DNA discovery.

Gift a kit

Refer friends, earn rewards.

Get reward

### ANCESTRY

- Ancestry Overview
- All Ancestry Reports
- Ancestry Composition
- DNA Relatives
- Order Your DNA Book

### HEALTH & TRAITS

- Health & Traits Overview
- All Health & Traits Reports
- My Health Action Plan
- Health Predisposition
- Pharmacogenetics
- Carrier Status
- Wellness
- Traits

### RESEARCH

- Research Overview
- Surveys and Studies
- Edit Answers
- Publications

### FAMILY & FRIENDS

- View all DNA Relatives
- Family Tree
- Your Connections
- GrandTree
- Advanced DNA Comparison



## Niemann-Pick Disease Type A

Niemann-Pick disease type A is a rare genetic disorder. It is characterized by an enlarged liver and spleen, developmental disability, recurring lung infections, and early death. A person must have two variants in the SMPD1 gene in order to have this condition.


[Overview](#)
[Scientific Details](#)

Niemann-Pick disease type A is caused by variants in the SMPD1 gene.

**SMPD1**




The SMPD1 gene contains instructions for making an enzyme called acid sphingomyelinase. This enzyme plays a role in converting a fat called sphingomyelin into another fat called ceramide. Certain variants in SMPD1 disrupt this function, causing a harmful buildup of sphingomyelin.

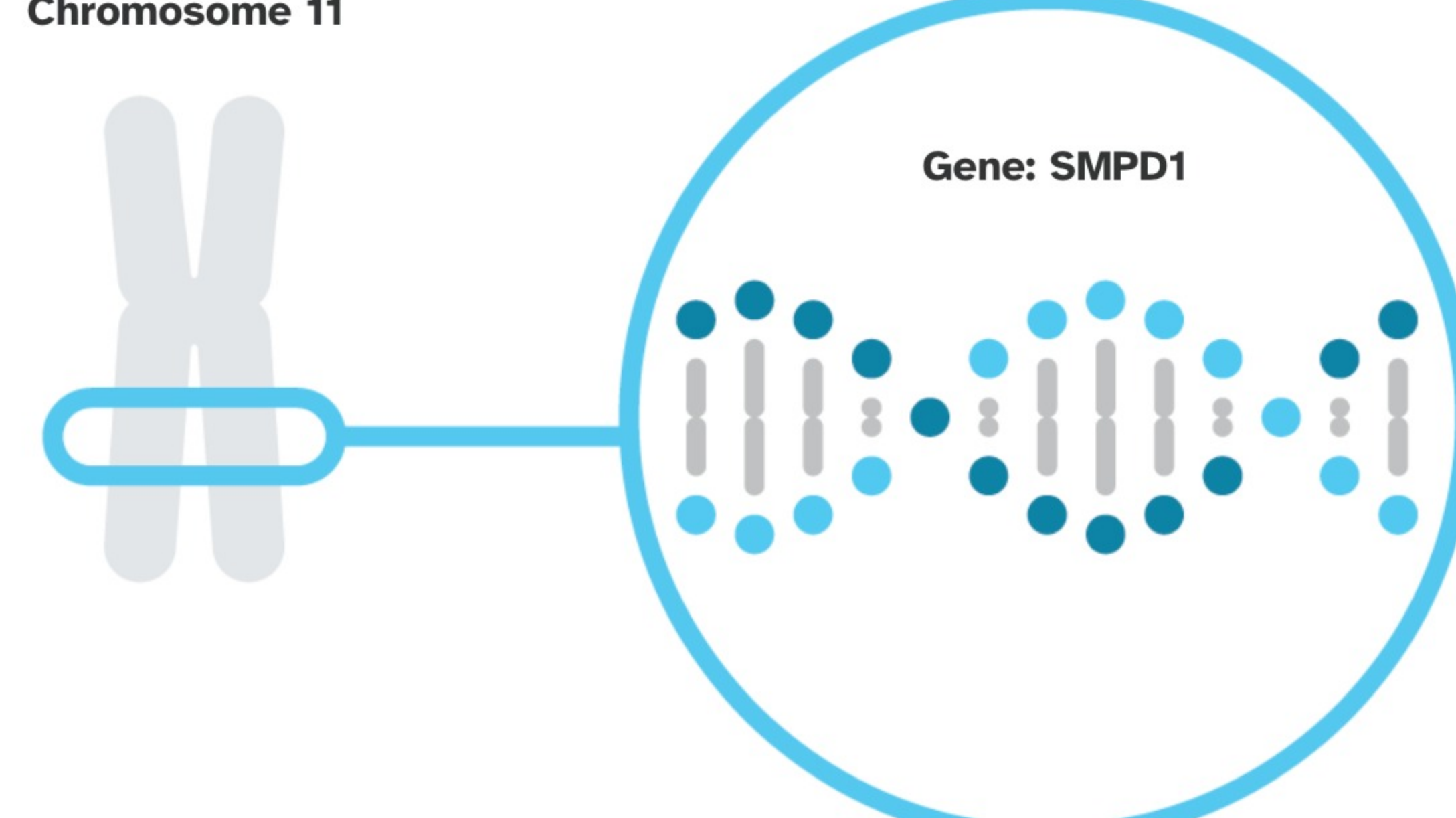Read more at [Genetics Home Reference](#)

**Chromosome 11**


You have no variants detected by this test.

[Variants Detected](#)
[View All Tested Markers](#)

Marker Tested	Your Genotype*	Additional Information
<b>L302P</b> Gene: SMPD1 Marker: <b>i4000381</b>	<b>T</b> Typical copy from one of your parents 	Typical copy from your other parent <ul style="list-style-type: none"> <li>Biological explanation</li> <li>Typical vs. variant DNA sequence(s)</li> <li>Percent of 23andMe customers with variant</li> <li>References [ 1 ]   <a href="#">ClinVar</a></li> </ul>
<b>fsP330</b> Gene: SMPD1 Marker: <b>i4000383</b>	<b>C</b> Typical copy from one of your parents 	Typical copy from your other parent <ul style="list-style-type: none"> <li>Biological explanation</li> <li>Typical vs. variant DNA sequence(s)</li> <li>Percent of 23andMe customers with variant</li> <li>References [ 1 ]   <a href="#">ClinVar</a></li> </ul>
<b>R496L</b> Gene: SMPD1 Marker: <b>i4000430</b>	<b>G</b> Typical copy from one of your parents 	Typical copy from your other parent <ul style="list-style-type: none"> <li>Biological explanation</li> <li>Typical vs. variant DNA sequence(s)</li> <li>Percent of 23andMe customers with variant</li> <li>References [ 1 ]   <a href="#">ClinVar</a></li> </ul>

\*This test cannot distinguish which copy you received from which parent. This test also cannot determine whether multiple variants, if detected, were inherited from only one parent or from both parents. This may impact how these variants are passed down.

23andMe always reports genotypes based on the 'positive' strand of the human genome reference sequence (build 37). Other sources sometimes report genotypes using the opposite strand.

## Test Interpretation

This report provides an estimate of the chances of still being a carrier for people who do not have the variant(s) tested. This is known as the **post-test carrier risk**.

Post-test carrier risk is based on the average chance of being a carrier for a given ethnicity and the carrier detection rate of the test for a given ethnicity.

[View technical article on estimating post-test carrier risk.](#)

### Post-Test Carrier Risk

This report provides an estimate of the post-test carrier risk for people of Ashkenazi Jewish descent only.

- For people of partial Ashkenazi Jewish descent, post-test carrier risk is less than that for those who are fully Ashkenazi Jewish. The exact post-test risk depends on how much Ashkenazi Jewish ancestry a person has.
- Post-test risk for other ethnicities cannot be provided because sufficient data is not available.

#### Post-test carrier risk for relevant ethnicities

Ashkenazi Jewish	1 in 3,000	[ 1 ]
------------------	------------	-------

## Test Details

### Indications for Use

The 23andMe PGS Carrier Status Test for Niemann-Pick Disease Type A is indicated for the detection of three variants in the SMPD1 gene. This test is intended to be used to determine carrier status for Niemann-Pick disease type A in adults, but cannot determine if a person has two copies of a tested variant. The test is most relevant for people of Ashkenazi Jewish descent.

#### Special Considerations

- Carrier testing for Niemann-Pick disease type A is recommended by ACMG for people of Ashkenazi Jewish descent considering having children. This test includes the three variants recommended for testing by ACMG.

### Test Performance Summary

#### Carrier Detection Rate & Relevant Ethnicities

The "carrier detection rate" is an estimate of the percentage of carriers for this condition that would be identified by this test. Carrier detection rate differs by ethnicity and is provided only where sufficient data is available.

Ashkenazi Jewish	97%	[ 1 ]
------------------	-----	-------

#### Analytical Performance

Accuracy was determined by comparing results from this test with results from sequencing. Greater than 99% of test results were correct. While unlikely, this test may provide false positive or false negative results. For more details on the analytical performance of this test, refer to the package insert.

### Warnings and Limitations

- This test does not cover all variants that could cause this condition.\*
- This test does not diagnose any health conditions.
- Positive results in individuals whose ethnicities are not commonly associated with this condition may be incorrect. Individuals in this situation should consider genetic counseling and follow-up testing.
- Share results with your healthcare professional for any medical purposes.
- If you are concerned about your results, consult with a healthcare professional.

See the [Package Insert](#) for more details on use and performance of this test.

\* Variants not included in this test may be very rare, may not be available on our genotyping platform, or may not pass our testing standards.

## References

- Gross SJ et al. (2008). "Carrier screening in individuals of Ashkenazi Jewish descent." *Genet Med.* 10(1):54-6. ↗
- Wasserstein MP et al. (2006). "Acid Sphingomyelinase Deficiency." [Updated 2015 Jun 18]. ↗

## Change Log

Your report may occasionally be updated based on new information. This Change Log describes updates and revisions to this report.

Date	Change
Feb. 18, 2016	Due to improvements in data analysis, some customers who previously received a "Not Determined" result for one or more of the following genetic markers may see a genotype at these markers: i4000381, i4000430. This may also update the overall report for these customers.
Oct. 21, 2015	Niemann-Pick Disease Type A report created.



Give the gift of DNA discovery.

Gift a kit

Refer friends, earn rewards.

Get reward

#### ANCESTRY

Ancestry Overview  
All Ancestry Reports  
DNA Composition  
DNA Relatives  
Order Your DNA Book

#### HEALTH & TRAITS

Health & Traits Overview  
All Health & Traits Reports  
My Health Action Plan  
Health Predisposition  
Pharmacogenetics  
Carrier Status  
Wellness  
Traits

#### RESEARCH

Research Overview  
Surveys and Studies  
Edit Answers  
Publications

#### FAMILY & FRIENDS

View all DNA Relatives  
Family Tree  
Your Connections  
GrandTree  
Advanced DNA Comparison